



ABOUT THE FYSIOLOGIC DIARY APP

Users of an ECG recorder (Holter) can use the Fysiologic Diary app to easily keep track of their complaints and activities in a diary. There is no longer any paper involved. Once the ECG recorder has been read, the data will automatically be sent to Fysiologic, where it will instantly be processed by our data experts. The app contributes to the speed and reliability of the data processing and offers convenience to the patient.

ABOUT FYSIOLOGIC ECG SERVICES

Fysiologic registers and analyses electrocardiograms on behalf of General Practitioners, specialists and hospitals. Information security is very important to us. The data you share with us via the app is sent to us via a secure connection and processed according to strict procedures. We are **ISO 27001 certified** and **NEN 7510 accredited**. For more information, please visit our website: www.fysiologic.com.

HOW DO YOU USE THE APP FOR PATIENTS?

1. DISTRIBUTION OF THE INFORMATION CARD

- Fysiologic makes information cards available which you can distribute among your patients, for example during an appointment or while waiting for an appointment.
- The card clearly explains how the Fysiologic Diary app works and what its benefits are.
- The patient will be well prepared for using the app after reading the card.

2. MAKING A QR CODE AVAILABLE

- The patient will need to scan a QR code in order to use the app. This can easily be created in the ECGAccess programme.
- Go to the prepared patient record in ECGAccess and click on **DIARY QR CODE**.
- Once the patient has downloaded the app, the QR code can immediately be scanned from the screen.
- You also have the option of printing and handing over or emailing the QR code to the patient.

3. AFTER USING THE ECG-RECORDER

- The patient's diary will automatically be closed once you have read the recorder and sent it to Fysiologic for further processing.